

Viraclean®

Hospital Grade Disinfectant

AUST L 69000




Specifically for use on healthcare surfaces



Viraclean®

Hospital Grade Disinfectant

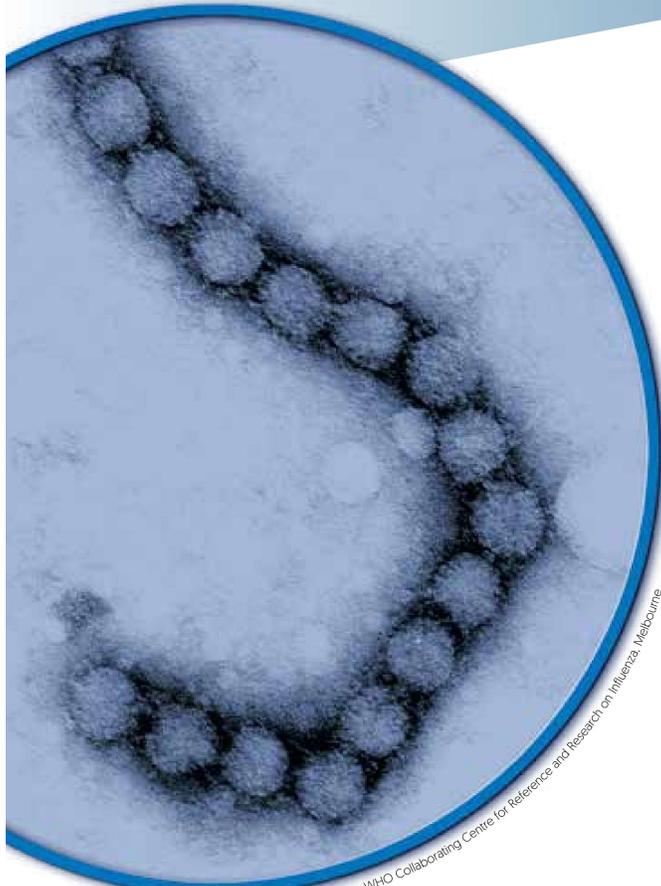


* Picture of influenza virus courtesy Ross Hamilton – WHO Collaborating Centre for Reference and Research on Influenza, Melbourne

Viraclean® is the result of years of intensive research into advanced cleaning and disinfecting technology from Whiteley.

Viraclean is proven to kill:

- ✓ Acinetobacter
- ✓ Candida albicans
- ✓ Coronaviruses including SARS-CoV-2 [COVID-19]
- ✓ Enterococcus faecalis [VRE]
- ✓ Escherichia coli [E coli]
- ✓ Hepatitis B Group virus
- ✓ Herpes Simplex virus
- ✓ Influenza virus
- ✓ Klebsiella pneumoniae [CPE/CRE]
- ✓ Proteus vulgaris
- ✓ Pseudomonas aeruginosa
- ✓ Salmonella choleraesuis
- ✓ Staphylococcus aureus [MRSA or Golden Staph]

Independent Studies on Viraclean®

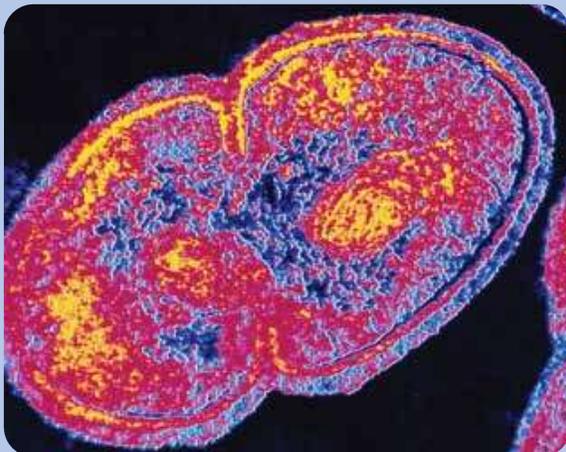

Viraclean® and V-Wipes have been proven to reduce the risk of environmental VRE contamination when used as part of a validated single stage cleaning/disinfection process, compared to a three stage process using a chlorine based disinfectant.*

*Please see: Friedman et al., Am J Infection Control:2013;41:227-231

Viraclean® can be used for both cleaning and disinfecting, demonstrating beneficial versatility for the healthcare and associated environs. Complex cleaning and disinfecting procedures can be instantly simplified by using Viraclean®.

Viraclean®:

- ✓ Preferred hospital grade disinfectant in Australia and New Zealand
- ✓ is non flammable
- ✓ is suitable for use on healthcare surfaces
- ✓ does not contain chlorine or oxidising biocides
- ✓ will not cause staining on fabrics or carpets
- ✓ has a neutral pH
- ✓ is friendly to metals (non corrosive)
- ✓ has a mild lemon fragrance
- ✓ suitable for use in both dirty and clean conditions

This product is not intended for use on therapeutic devices. Instrumax® Pink is available as an Instrument Disinfectant Low Level for use in disinfecting non critical medical devices.



Viraclean® Uses:	
Nursery/Childcare:	Cleaning cribs, cots & change tables.
Central Sterilising Department (CSSD):	Bench tops, stainless steel surfaces, sinks, trolleys, tap fittings.
Bathroom (toilet/shower):	Sinks, tiles and all hard surfaces.
Intensive Care Units:	Cleaning and disinfection of all hard surfaces.
Manual Cleaning of Theatre Equipment: (excludes medical devices)	Viraclean® can be used to clean and disinfect equipment, cables and will not adversely affect surfaces.
Operating Theatres:	Cleaning and disinfecting theatre surfaces.
Blood spills: (excludes medical devices)	Viraclean® is ideal for cleaning and disinfecting blood spills.
Telephones:	Viraclean® is ideal for cleaning and disinfecting phones where large numbers of bacteria are found.
Dental & GP surgeries: (excludes medical devices)	Viraclean® can be used in Dental & GP surgeries for cleaning and disinfecting surfaces including chairs, stools, tables benches and light fittings.
Laboratories & Pathology clinics:	Viraclean® is excellent for wiping down benches, disinfection of white rooms, cleaning and disinfection of fume cupboards etc.
General high touch areas:	Desks, door handles, railings and work stations.

Viraclean® Chemical Composition and Properties

Surfactant Blend	Blended
Colour	Light pink
Colour of Dilute Solution	Colourless
pH	Neutral [7]
Solubility	100% in all water
Environmental	Completely biodegradable
Specific Gravity	0.99
Corrosion inhibitor	Not necessary
Foaming Properties	Slight
Occupational Safety	Use as spray or liquid
Dangerous Goods Rating	Not applicable
Shelf Life	18 months
Safety Equipment	Use of gloves and eye protection is recommended.

Insist on Whiteley when ordering your Infection Prevention products

HEALTH STATEMENT

Whiteley makes every effort to ensure that its products are always safe to use, however products can affect individuals differently and the company cannot predict individual responses. Therefore careful reading of the Viraclean® SDS is essential prior to first use.

ENVIRONMENTAL STATEMENT

Whiteley manufactures products that comply with Australian and International environmental policy. Whiteley aims to ensure that every product is 100% biodegradable, whilst achieving optimal performance and benefit to customers.

All Whiteley products are supplied in 100% recyclable containers.

DISTRIBUTED BY:



ABN No. 24 000 906 678
PO Box 1076, North Sydney NSW 2059
Product Support Hotline: 1800 833 566
www.whiteley.com.au

NZBN: 9429041224243
PO Box 22-519, Otahuhu, Auckland 1640
Product Support Hotline: 0800 257 352
www.whiteley.co.nz

Whiteley 

Protecting lives and livelihoods since 1933